

Core · it® Dual Technical Guide




Core Build-up Material Technical Guide



1. Use EDTA (Do not use H₂O₂) to chemically clean the post space.



2. Clean and rinse the post space, then thoroughly dry using paper points.

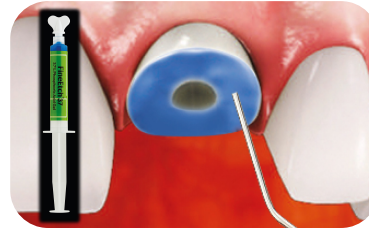

3. Etch according to your preferred technique.



4. Rinse and Dry.



5. Mix adhesive and activator, apply it in the root canal and dentin.



6. Prepare the fiber post according to the manufacturer's instructions.




7. Dry with air.



8. Apply Core-it® Dual directly into the post space.



9. Insert the post immediately into the post space within 1 min after Core-it® Dual application.

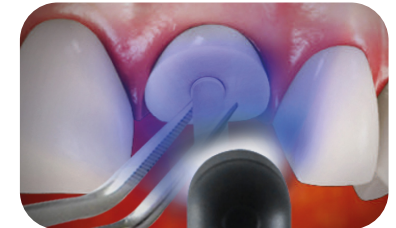


10. Light cure each surfaces for 20 seconds (LED 1200mW /cm²).



11. Let the material set for 4 minutes.



12-1. Apply Core-it® Dual directly into the matrix.

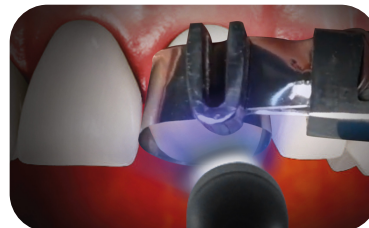

12-2. Light cure (20 sec), or Self curing (4min).



13. Tooth preparation.



14. After treatment.