Phosphoric Acid Gel

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Conformity with	CE Comunaute Europeenne
Temperature limit	2°C 27°C
Keep away from sunlight	
Keep dry	
Authorized representative in the European Community	EC REP
Federal law prohibits dispensing without a prescription	Rx Only

Batch code	LOT
Date of manufacture	
Used-by date	
Caution	
Manufacturer	
Medical Device	MD

Phosphoric Acid Gel

General Information

FineEtch® - 10, 37 is phosphoric acid (H_3PO_4) semi-gel-type agent intended for use on dentin and enamel. FineEtch® will effectively remove the smear layer and help successful bonding.

Composition

- Distilled water, Phosphoric acid, Pigment, etc.

Indication

- FineEtch® is a pre treatment agent which helps bonding agent having a strong bonding strength between resin and teeth.

Shade

- Blue

Precautionary Information

1. Check for any apparent deformation, discoloration, cracks, or foreign matter.
2. Check the cleanliness of the product before use.
3. Do not use products that appear to be unpacked, damaged or discolored.

Application

1. After changing the cap of the syringe to a tip, clean and dry the cavity.
2. In the case of resin and acrylic temporary restoration, make sure to completely remove of the existing amalgam or composite resin.
3. Etch entire cavity with FineEtch®. (Enamel: approx.20~30sec., dentin:max 15sec.)
4. Rinse with clean water and dry with air.
(Caution: Remove excess with air blower and leave moist in dentin surface.
If surface is not chalk-white/matt appearance, repeat this etching step.)
5. After the procedure is completed, remove the tip and close the cap for the next use.
6. Follow the manufacturer's instructions, regarding self-priming adhesive and restorative materials application and curing.

Caution

1. FineEtch® is designed for sale and use by licensed dental care professionals only.
It is not designed for sale nor is it suitable for use by non-dental care professionals.
2. FineEtch® is corrosive and contains phosphoric acid.
3. If FineEtch® causes an allergic reaction or oversensitivity, discontinue its use immediately.

Manufacturer

SPIDENT CO., LTD. www.spident.co.kr
203 & 312, Korea Industrial Complex, 722, Gojan-Dong,
Namdong-Gu, Incheon, Korea 405-821
TEL. +82-32-821-0071 FAX. +82-32-821-0074

EU representative

AR EXPERTS B.V. (tradename : CE Medical)
Amerlandseweg 7, 3621 ZC, Breukelen, the Netherlands
TEL. +31 (0)85 007 32 20

4. Avoid contact of FineEtch® with eyes, mucosal membrane, skin and clothing.
5. If FineEtch® comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
6. For dental use only.
7. Do not use FineEtch® after date of expiration.
8. Tip for single use only, do not reuse.

Storage & Use

1. Do not expose restorative materials to elevated temperatures, or to intense light.
2. Store in shady and cool place.
3. Shelf life at room temperature is 3 years.
4. Avoid contact with air.

Warranty

Spident® warrants this product will be free from defects in material and manufacture. Spident®, MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Spident®'s sole obligation shall be repair or replacement of the Spident®'s product.

Limitation of Liability

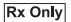
Except where prohibited by law, Spident® will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet. Caution: U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

Disposal

It should be handled according to local laws and regulations.

Conformity with	 Comunaute Europeenne
Temperature limit	
Keep away from sunlight	
Keep dry	
Authorized representative in the European Community	
Federal law prohibits dispensing without a prescription	

Batch code	
Date of manufacture	
Used-by date	
Caution	
Manufacturer	
Medical Device	