

Temp · it® & Temp · it® Flow Technical Guide



Clinical Case _ Temporary fillings




1. Preparation



2. Apply Temp-it with a suitable instrument or apply dispensing tips for Temp-it Flow.




3. Apply Temp-it / Temp-it Flow into the cavity.

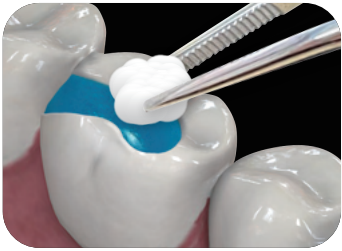

4. Use alcohol-soaked cotton pellet to contour.



5. Check occlusion.

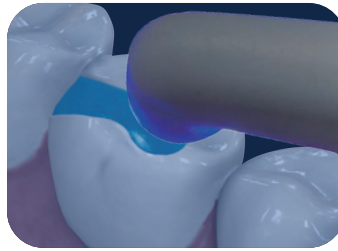

6. Light cure for 10-20 seconds (depending on layer thickness).



7. Recommended use no more than 2 week.




8. Remove in one piece when final restoration is ready.


Clinical Case _ Implant Abutment Sealing



1. Abutment with open aperture before final restoration.

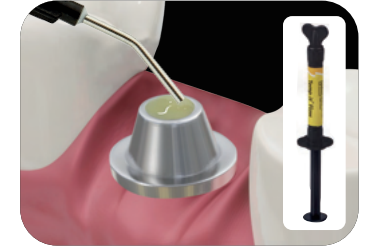

2. Apply Temp-it Flow to seal the aperture temporarily.

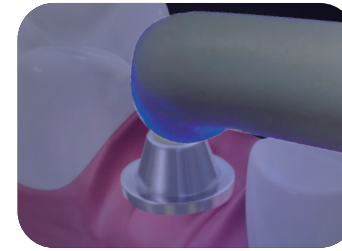

3. Light cure for 10-20 seconds.



4. Seat the crown over the abutment.



5. Remove easily in one piece if necessary.