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Conformity with	CE Comunaute Europeenne
Temperature limit	23°C
Keep away from sunlight	
Keep dry	
Authorized representative in the European Community	EC REP
Federal law prohibits dispensing without a prescription	Rx Only

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Batch code	LOT
Date of manufacture	
Used-by date	
Caution	
Manufacturer	
Medical Device	MD

## Light-Cured Temporary Filling Material

### General Information

Temp-it® is a light-cured provisional filling material for temporary restorations. It keeps maintaining the shape of the tooth and the masticatory force until the permanent material is restored. It has three colors (yellow, blue, sky blue). It can be used as temporary restorations in any part of the inlay and onlay. It is also easy to be used because it is not sticky to the spatula and can be easily removed in one piece.

### Composition

- Organic Matrix : UDMA, HEMA
- Inorganic Fillers : Nano silica
- Additives : Initiator, Catalyst, Stabilizer, Others

### Indications

- Temporary filling of Inlay / Onlay
- Temporary restoration / Sealing of dental caries
- Sealing of implant screw access cavity

### Shade

Yellow, Blue, Sky blue

### Precautionary Information

1. Check the expiration date and avoid package contamination.
2. Follow the instructions for use before using it.
3. Do not use if the material has been cured.

### Application

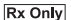
1. Finish the pretreatment of an inlay, onlay or the place where temporary restoration is required
2. Carefully clean and dry the cavity to be restored before applying Temp-it®.
3. Place a sufficient quantity of Temp-it® into the cavity using a spatula or another suitable instrument.
4. Remove excess material around the teeth.
5. Light cure for 10–20 seconds with a curing light.

Light Intensity (mW/cm <sup>2</sup> )	Exposure Time (sec)	Depth of cure (mm)
≥ 500	20	4
≥ 1000	10	4

6. Check occlusion and adjust it using check-bite.

### Caution

1. Temp-it® should be used by dentists and doctors, observing the scope of application, instructions and precautions.
2. Avoid the use of Phenolic substances of other phenolic containing materials in combination with Temp-it® because Phenolic substances, especially preparations containing eugenol or thymol or clove oil, interfere with the curing of Temp-it®.

Conformity with	 Comunaute Europeenne
Temperature limit	 23°C
Keep away from sunlight	
Keep dry	
Authorized representative in the European Community	
Federal law prohibits dispensing without a prescription	

### Manufacturer

SPIDENT CO., LTD. [www.spident.co.kr](http://www.spident.co.kr)  
203 & 312, Korea Industrial Complex, 722, Gojan-Dong,  
Namdong-Gu, Incheon, Korea 405-821  
TEL. +82-32-821-0071 FAX. +82-32-821-0074

### EU representative

AR EXPERTS B.V. ( tradename : CE Medical )  
Amerlandseweg 7, 3621 ZC, Breukelen, the Netherlands  
TEL. +31 (0)85 007 32 20

3. Temp-it® contains substances that may cause an allergic reaction by skin contact in certain patients
4. Do not use to patients with histories of hypersensitivity such as rash and dermatitis in Temp-it®.
5. Stop applying to the patients with histories of hypersensitivity such as rash and dermatitis in Temp-it®.
6. Be careful not to contact with eyes and use only to the oral tissue.
7. Wash immediately eyes or skin with plenty of water and then consult a doctor, if they contact with materials before polymerization.
8. Make sure to use protective devices for the respiratory tract before surgery.
9. Do not use after expiration date (3 years).
10. If refrigerated storage is used, allow product to reach room temperature prior to use.

### Storage & Use

1. Store Temp-it® in a dry and cool place. (Temperature: 2–27°C)
2. Close the lid immediately after use.
3. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life.
4. Avoid high temperature and strong light.
5. Expiration date is 3 years from the date of manufacture.
6. See outer package for expiration date.
7. Do not store materials in proximity to products containing Eugenol or thymol or clove oil.

### Warranty

Spident® warrants this product will be free from defects in material and manufacture. Spident®, MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Spident®'s sole obligation shall be repair or replacement of the Spident®'s product.

### Limitation of Liability

Except where prohibited by law, Spident® will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

### Disposal

It should be handled according to local laws and regulations.

Batch code	
Date of manufacture	
Used-by date	
Caution	
Manufacturer	
Medical Device	